


GUS & GEAR®

LOCK 3000 Belt Instructions



STEP 1:

- Completely open the snaps and velcro on the front flap. Optional: Open the velcro tab near the circular opening.

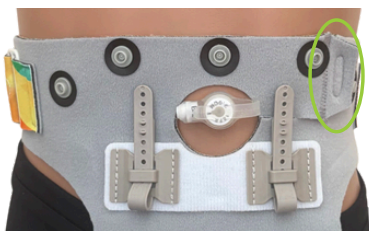

- Thread the belt through one side of the tri-glide, velcro side facing outward.
- Fold over to secure with the velcro tab.



- Place the circular opening around the G-tube or other device.
- Bring all tubing through the opening and center the device in the opening.
- Bring the second end of the belt around the back and thread it through the other tri-glide. Close the velcro tab if it has been opened.



- Place the Dual Safe Strip directly below the circular opening and firmly press to attach.




- Adjust either end of the belt for a snug fit and secure the velcro. By adjusting both ends of the belt, the velcro closures can be accessible or hidden.



- If using a low-profile feeding tube (button) with the extension set disconnected, proceed to Step 3.

STEP 2:

- Secure the G-tube extension set or PD catheter by coiling the tubing as shown.
- Place the connector between the two straps on the Dual Safe Strip.
- Use one strap to secure the device and the other to secure the tubing.
- Thread the tubing in any direction to exit the front flap area.

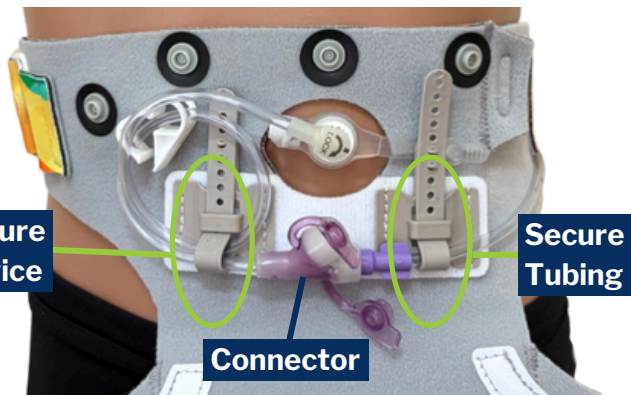

STEP 3:

- Close the front flap by engaging the snaps and velcro.



- Optional: Secure the tubing to the belt using a Single Safe Strip.



CARE INSTRUCTIONS

Close all Velcro prior to washing.
Machine wash gentle or hand wash.
Cold water. Hang dry only.



IFU & Spanish instructions available at GusGear.com


GUS GEAR®

